



EXPLORE MORE

Scan the QR code to check out Angelus' full line of periodontal products.



FOR FREE INFORMATION:

800.921.4806 ext. 359
www.dps.li/a/6B9-1146

ANGELUS

BIO-C TEMP

Because teeth are under constant stress, the need for clinicians to find materials that can withstand the onslaught of various forces and help keep the tooth sealed is of utmost importance. Although the definition of a "bioactive material" is still evolving, the liners, composites, and cements available today are making great strides toward helping recharge and remineralize tooth structure in a way that prevents restoration-threatening secondary caries.

Angelus' BIO-C TEMP was developed with a formulation that provides a gradual and slow release of its main components, which encourages a healthy restoration of the affected region. Unlike calcium hydroxide pastes, which require frequent replacements, BIO-C TEMP has low solubility that allows the product to remain in contact with the root canal over longer periods of time. As a result, it

provides continuous and gradual release of the hydroxyl ions that are responsible for an increase in pH, while eliminating the need for new applications.

BIO-C TEMP is indicated for use as an intracanal dressing in a variety of scenarios, including during endodontic treatment in teeth with pulp necrosis and retreatments, or in teeth with perforations and external/internal resorptions. It can also be used prior to applying MTA ANGELUS or MTA REPAIR HP, and to treat of incomplete root formation.



TOP 3 REASONS TO USE BIO-C TEMP IN YOUR PRACTICE

- 1 Biocompatible product ready for use
- 2 High alkalinity
- 3 Excellent radiopacity

STEPS TO PERFORM BIO-C TEMP APPLICATION:

- Insert the tip of the syringe near to the dental apex
- Apply BIO-C TEMP by gradually withdrawing the syringe to obtain complete filling of the canals
- Seal the coronary opening with materials that promote a good seal



CATEGORY INDEX

MANUFACTURER	PAGE		
ACTEON North America	XX	Dia-Dent Group International	NSK
Angelus		Henry Schein Dental	SS White
Brasseler USA		iTENA North America	Ultradent Products, Inc
COLTENE		JS Dental Manufacturing, Inc	Vista Dental Products
Dentazon Corporation		KaVo Kerr	
Dentsply Sirona		Medidenta	