PINJET

PORTUGUÊS

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ENGLISH

DEFINITION: Polycarbonate post for intra-root impression and/or build-up. **COMPOSITION:** Polycarbonate. **INDICATIONS:** Root canal impression for the fabrication of metal posts indirectly; Post molding for the fabrication of metal posts directly. **TECHNIQUE OF USE:** **A. INDIRECT ROOT CANAL IMPRESSION** - 1. After canal preparation, adapt PINJET loosely in canal (there should be a space between pin and canal wall); 2. Apply on PINJET the specific adhesive of the selected impression material; 3. Apply the impression material on PINJET and place it in root canal. **B. DIRECT METAL POST MOLDING** - 1. After canal preparation, apply a separating agent on canal; 2. Reline PINJET with self-cured acrylic resin in the canal; 3. Build core of post with self-cured acrylic resin; 4. Make final adjustments and send mold to lab for casting.

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1050500 - 0418052023

NOTE: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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CEP 86031-218 - Londrina - PR - Brasil

For the digital file, please visit our website through the link in the QR code. ▼

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ATTENTION: This product must be used according to the instructions described in this manual. The manufacturer is not responsible for failure or damage caused by incorrect handling or use.

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